# Eporex K69

## EPOREX - ESTETICA

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## EPOREX - ESTHETICS

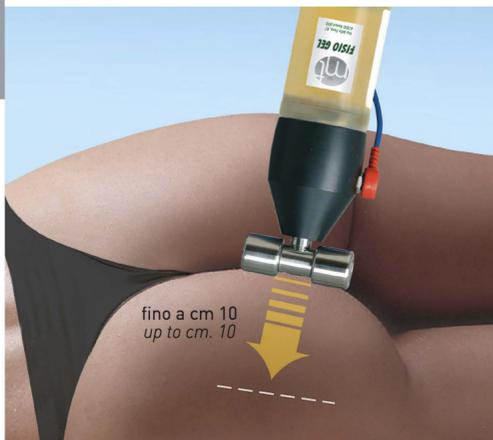
*Drastic diets, strenuous physical exercise, prolonged, painful treatments: all this to counteract decreased skin tone, ageing, stretch marks and cellulite. Millions of women go through these trials and tribulations every day, often with unsatisfactory results, particularly in terms of skin tone and elasticity.*

*EPOREX K69 treatment, on the other hand, ensures good results from the very first sessions. It can treat cellulite, improve skin and muscle tone, make stretch-marks less evident, improve facial tone, moisturize and even reduce wrinkles; all without needles or any personal discomfort. It is a fine-tuned, pleasant treatment able to give your skin everything it needs.*

*EPOREX K69 makes the most of electrophoration. Based on ionophoresis, EPOREX K69 exploits the principles of mutually enhanced, elasto-modulated waves to deliver active ingredients to the skin. EPOREX K69 differs from classical ionophoresis in that it uses elasto-pulsating waves of higher capacity and intensity able to deliver more active ingredients deeply with greater respect for sensitive skin.*

## ACTIVE TRANSDERMICAL TRANSMISSION

The Isoforesis technological device EPOREX K69 allows the active penetration of high and low molecular weight substances through the dermo - epidermal barrier, without the substance sustains any damage.

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Power Supply: 90V, 220V c.a.  
Frequency: 50Hz, 60Hz  
External Power Supply: 24Vcc  
Absorbed power: 40VA  
Protection: 2 x R 2A 250V  
Security class: I BF  
Max Power Out: 45V  
Corrente massima di uscita (R=1K): 45mA  
Frequency Max: 1760Hz  
Net weight: 6 Kg.  
Dimensions: 550x260x.230 mm.

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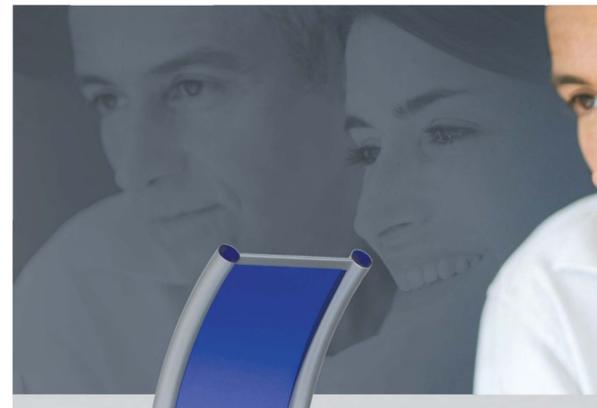
Health & Beauty Station  
Needle Free Mesotherapy

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## METHODOLOGY

*The active transmission is effected without any pain for the patient, by the emission of low intensity elastic pulsed waves, that are able to active electronically high and low molecular weight allowing the passage in the deeply extra-cell compartment. The trans cutaneous transport take place because of the peculiar connection of the elastic pulsed wave and the ionisation chamber, where the active principle mixed before with a specific gel become electrically "charged" and ready for the ionic transmission.*

## BENEFITS

- Transmission of the active principle directly in the extra-cell compartment, where the obtained high concentrations active quickly the metabolism of the target cell that thank to the osmotic gradient catch and utilize the transmitted substances.
- The active principle non-absorption of the vascular compartment, with no systemic toxicity, especially in the usage of active principle particular typology
- Possibility to transmit every molecular typology, also macromolecule, skin is notoriously proof of it, avoiding painful administration method.
- Total absence of any type of pain or collateral effect
- Quickness results in aesthetical and medical field.
- Methodology adaptability to several application in the aesthetic field.
- Innovative in comparison to ionophoresis as the ISOFORESIS gives the possibility to choose the depth from cm 0,2 to cm 10 to make the substance penetrate with a high mixture percentage in the required zone. (90% against the 5% of traditional device).

## TARGETS AND RESULTS

- The methodology versatility allows to obtain very good results in firming up, nutrition, revitalizing, increase tissue elasticity and the pain therapy in several inflammatory pathology.*
- Cellulite
  - Stretch mark
  - Muscular modelling
  - Acne
  - Wrinkles (Anti aging)
  - Face lift
  - Localized Inflammation caused by sprained muscle, luxation, straining, contusions
  - Baldness stop
  - Fat

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### WHAT IT IS AND HOW IT WORKS

*EPUREX K69 is a pulse-modulated ionic current generator that ensures totally painless, non-invasive transdermal electrostimulation. A special handset - with an ionization chamber attached to a small container holding the specific conducting gel - is connected to the unit. A soft rubber plaque interfaces with the skin. The appropriately modulated pulsating current is applied at the point where the handset comes into contact with the skin. In this way, the most appropriate active ingredients for treating the problem in question are transferred to the skin itself. Control of the ionic current wave shape and the treatment is immediate and intuitive, thanks to microprocessor for customized treatment.*

### HOW TO USE IT

*This unit is easy to use and intuitive. A menu on the front of the unit is used to select a specific program depending on the type of blemish to be treated. Then place the small rubber plaque near the area to be treated and connect a second electrode to the special EPUREX K69 handset. The Eporex K69 handset is designed to ensure total effectiveness. It ensures excellent ionization of the various molecules present in the gels available. Once ionized, a special stainless steel roller keeps the active ingredients ionized until they are released from the hollow cavity in the roller and come into contact with the skin. Once in contact with the skin, the difference in electric potential between the handset and the plaque, created by a constant capacity chamber, ensures that the various substances pass into the skin where they perform their action. Eporex K69 has two handsets: one specific for the body, with two small metal rollers designed for treatment of larger areas and a second, smaller handset for facial treatments. The parts that come into contact with the patient are made of stainless steel, so they can be sterilized after each treatment.*

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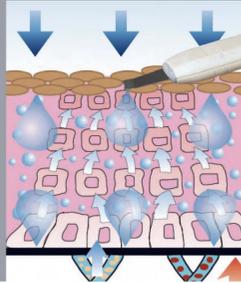
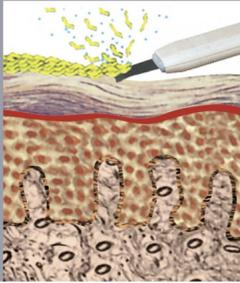
*Skin ageing does not depend only on "chronological" age but is often accentuated by environmental factors, harmful substances, medicines, smoking, stress and an incorrect approach to exposure to UV rays. Keeping skin healthy in terms of both appearance and function also depends on complex interactions:*

- **Factors stimulating skin regeneration**
- **Factors inhibiting excessive cell growth (chaloni)**
- **Enzyme and cell mechanisms repairing local damage.**

*These mechanisms must always be active and well-balanced. Imbalances between these factors cause more or less deep wrinkles, thickening of the skin, senile pigmentation and hyperkeratosis and a loss of skin moisturising and elasticity, and are also detrimental for the micro-circulation, with consequent lower skin nutrition and toxin-elimination which is damaging to the connective fibres (structures that retain water in the dermis to ensure stability and elasticity).*

### ULTRA-SOUND

*Today, action can be taken against these blemishes with BRIGHT SKIN, a professional tool that helps rejuvenate skin and prevent skin ageing through ultra-sound waves. A special control knob (transducer) and a piezoelectric system oscillating at a frequency of 25,000 Hz exploit the cavitation phenomenon. The control knob comprises a spatula-shaped metal blade folded at the end by 45°; this angle is fundamental in ensuring that the ultra-sound wave is focused over the entire surface and to improve the process of lifting and detaching dead cells from the surface of the skin, as well as to ensure that the blade itself slides perfectly over the skin.*

**Thanks to the synergy of the different systems used, helps achieve excellent results in skin care, for younger, more elastic, brighter and healthier skin.**

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### CAVITATION

*This phenomenon occurs when a liquid is subjected to a high vacuum. As soon as the absolute pressure becomes lower than the tension of the liquid, an force energy field is created with significant conversion into steam in the form of micro-bubbles of the gases contained in the liquids, that in turn encourages the detachment of dead cells through "thrust", at the same time as encapsulating residues of make-up and impurities deposited in sebum follicles. The same blade also combines (in relation to the selected program) the issue of galvanic micro-currents for electronic delivery or peeling, thus exploiting an important synergy between the action of the ultra-sounds (Sonic thrust) and the micro-currents (Ionic Thrust).*

*The action mechanism of BRIGHT SKIN is highly effective in the different application techniques used to treat the most common blemishes:*

### PEELING

*Sonic cavitation removes the corneal stratum and surface cells to release a stratum of younger corneocytes (cell turn-over), at the same time as supplying galvanic micro-currents that provide an important additional ionic action for skin tone and luminosity.*

### EXOGENOUS AND ENDOGENOUS MOISTURISING

*Micronise the active ingredients and delivers them at depth through sonic thrust; thanks also to the osmotic phenomena activated by the compression and decompression effect on interstitial liquids, major improvement in skin moisturising is also achieved.*

### EXOGENOUS AND ENDOGENOUS MOISTURISING

*Micronise the active ingredients and delivers them at depth through sonic thrust; thanks also to the osmotic phenomena activated by the compression and decompression effect on interstitial liquids, major improvement in skin moisturising is also achieved.*

### RE-BALANCING THE METABOLISM

*Improves micro-circulation with increased tissue oxygenation stimulating the production of collagen and elastin through a micro-massage with elasto-pulsed waves.*

### DERMO-PURIFYING ACTION

*Re-balances the function of the skin's hydro-lipidic film.*

### WHITENING ACTION

*Combined with a specific product, it reduces the blemishes caused by melanin discolouring and enhances the anti-oxidant mechanisms.*

### TONE-UP MICRO-MASSAGE

*to re-activate and stimulate skin structures and cell activity. BRIGHT SKIN, thanks to the synergy of the different systems used, helps achieve excellent results in skin care, for younger, more elastic, brighter and healthier skin.*